


PROPER FITTING PROCEDURE

StrideEquus

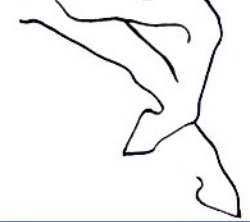

4 EASY STEPS

marquis®
supergrip

1 SLIDE HOOF INTO THE OPEN BOOT

Quick Check List

- * Start with a clean hoof and boot.
- * Open the boot wide.
- * Lift up the hoof.
- * Slide the clean hoof into the boot - toe first.

Are you slowly introducing your horse to the boots over a few 20 minute sessions?

Are the hooves and boots clean?

Is the hoof seated down into the bottom of the boot?

Is the toe to the front of the boot?

The hoof is centered inside the boot?

2 CHECK THE POSITION OF THE HOOF

- * Tap the hoof on the ground as the horse steps down into the boot.
- * Check to see that the hoof is all the way to the front of the boot.
- * Open the sides of the boot to check that there is not more than a 1/2 cm or 3/16" space on either side between the hoof wall and boot wall.

Boots on the correct feet - the clip is to the outside of each boot?

Wire is securely fixed?

Is the clip closed and secure?

3 FASTEN THE BOOT CLOSED

- * Affix the wire on the inside of the boot to your choice of setting.
- * Close the clip on the outside so that the front flap is snug onto the hoof wall.
- * Slide your finger down the back of the boot between the air chamber and your horse's bulbs. It should feel snug but not tight or loose.

The boot is the correct size/fitting correctly in the sides and back?

Air chamber is filled to the desired fullness?

Valve caps in place?

4 PUMP UP THE AIR CHAMBER

- * Squeeze pump onto valve by pressing between your thumb and forefinger. If air is escaping there will be a hissing sound - just re-position the pump squarely onto the valve. Release the pump from the valve quickly so that no air escapes (a slight flick of the wrist may help).
- * Check the inflated air chamber for firmness - it should be firm with give and not hard.

Questions? Call us at 1-800-403-0689